

# 140x210mm



**Dr. Odin<sup>®</sup>**  
**Oxygen Concentrator**

**Instruction for Use:**

**Model: JMC5A Ni**

- 1 Use a good quality 1.5Kw voltage stabilizer
- 2 Keep the concentrator 1-2 ft. away from wall, both Left and Back Side
- 3 Avoid using machine in dusty environment
- 4 It is compulsory to get regular performance checks done in every 3 months (AMC or CMC)
- 5 Do not run the machine continuously for long durations. Give it some rest after 6-7 hours for minimum 30 minutes
- 6 Use clean distilled/RO water in humidifier and replace it every 2 days
- 7 Unit outer cabinet should be clean with a mild household cleaner
- 8 Unit filters should be cleaned once in a week and replaced after every 3 months
- 9 If you want to run the oxygen concentrator on inverter, then inverter should be compatible

All the above stated measures will improve the life of the oxygen concentrator and reduce your expenditure on repair/service of machine.

Note: \*In case the above instruction are not followed the company is not responsible for any kind of breakdown during and after warranty period. In such case all spare parts, repairable works and services will be chargeable as per company norms.

However, a concentrator can still break down anytime as after all it is a machine. Therefore, it is highly recommended to keep a standby cylinder with you always because a service engineer would also take some time to reach your home to repair the concentrator. One should ideally keep a backup cylinder that would last for 24 hours.